
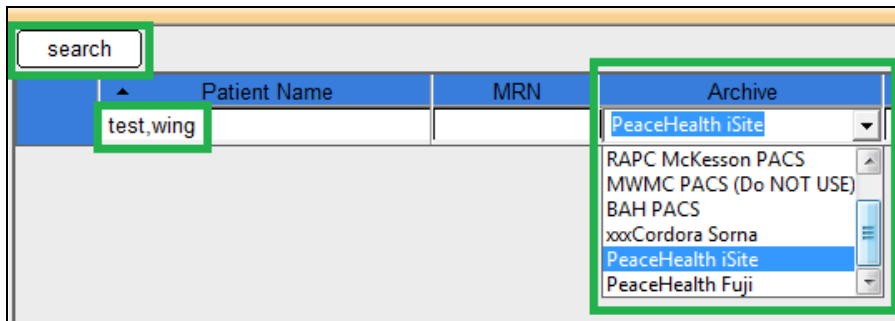


iQuery – DICOM Query/Retrieve Studies From Other PACS

iQuery is the iSite feature that performs DICOM Query/Retrieve against other PACS systems. Some of the facilities in RAPC’s service areas have agreed to allow our PACS users to query and retrieve studies from their PACS as a more efficient way to get needed studies. iQuery is not available to all PACS users.

Query

- Select ‘iQuery’ from the Folder List
- Select the ‘Archive’ from the drop down list
- It is not recommended searching ‘All’ as the search may take a long time.
- Enter the patient’s name **without a space after the comma** and click ‘Search’

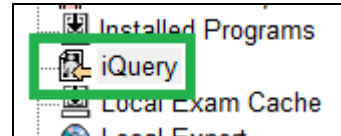

Patients matching your query will be returned in the list.

Retrieve


Expand the patients list of studies and select the desired study/studies, hold Ctrl button for multi-select. Right click on a selected entry and click ‘Retrieve’

Study Description	MRN	PACS	Timestamp
TEST, Wingding, A	02229217	PeaceHealth iSite	
Study Description: SCRAPBOOK			07/14/2005 09:46:38
Study Description:			12/22/2006 07:39:25
Study Description:			12/22/2006 07:39:25
Study Description: TEST MRCP			01/29/2007 14:25:32
Study Description: KNEE^LEFT 4 1			04/07/2007 10:58:49
Study Description: KNEE^LEF			04/07/2007 10:58:49
Study Description:			05/04/2007 07:42:55
Study Description: OPI ANKLE RIGHT 3V OR MO...			05/19/2007 02:32:31
Study Description: US ABDOMEN COMPLETE			06/11/2007 09:45:00
Study Description:			08/02/2007 14:35:04
Study Description:			08/10/2007 12:54:19

The 'iQuery Confirmation' window will appear.

A summary of the selected patient and studies will be in the top of the window

You may select 'STAT' to hasten the retrieval if the sending PACS is able to modify priority.

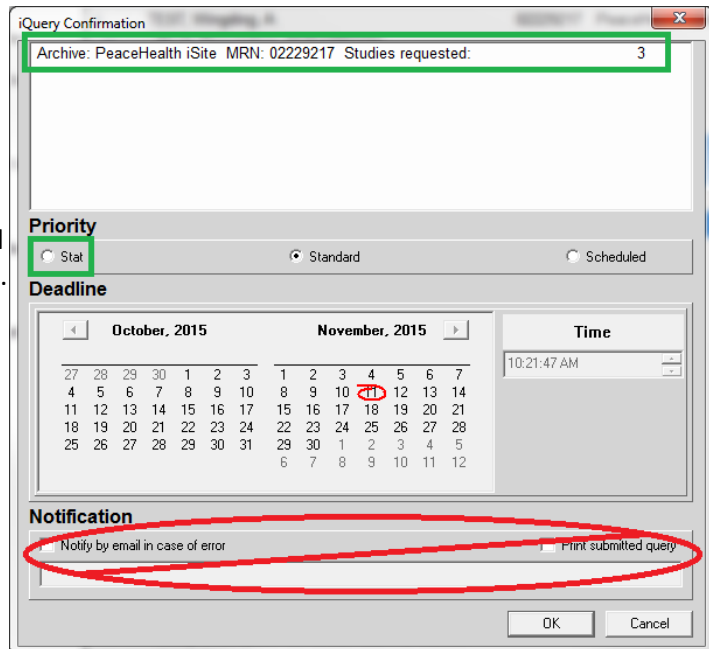

The "Notification" features are not functional

The retrieved studies will be exceptions since the MRN and Accession of the study are not known to the PACS. Exceptions can be viewed in Exceptions Lookup or Patient Lookup.

Organiz...	Patient Name	Study Description	MRN	Accession #
test, wing				
RAPC	TEST, Wingding, A	OPI ANKLE RIGHT 3V OR MORE	02229217	3635276
RAPC	TEST, Wingding, A	OPI ANKLE RIGHT 3V OR MORE	02229217	3635276
RAPC	TEST, Wingding, A	KNEE^LEFT 4.1	02229217	4238834
RAPC	TEST, Wingding, A	KNEE^LEFT 4.1	02229217	4238864

If the studies are needed as priors for comparison, 'Create Exam in Exception Handler' can be utilized, please see "ISP-CreateExamExceptionHandler" documentation. Please ask RAPC IT for this document. Not all users have the ability to create exams in Exception Handler.