

Linking Patients Version 4.4.540

In the latest version of Intellispace PACS Patient Linking looks different than version 4.4.526. The 'Tree Structure' with a Master Record and expanding [+] layers of Organization Records has been replaced with one list of exams from all orgs combined together.

Patient Name		MRN	Date of Birth	Organization	
test, adult					
	TEST, ADULT	10004	1/1/1974	WMC	M
	No Descriptions	Acc: 5281	Date: 2/26/2015 12:13:00	MRN: 10004	Org: WMC
	No Descriptions	Acc: 5245	Date: 2/3/2015 16:05:00	MRN: 10004	Org: CHS
	No Descriptions	Acc: 5244	Date: 2/3/2015 16:01:00	MRN: 10004	Org: CHS
	MR - MR Cardiac Function/Morp...	Acc: 3634	Date: 10/15/2014 10:00:00	MRN: 10004	Org: MRIA
	No Descriptions	Acc: 2401	Date: 9/19/2014 7:00:00	MRN: 10004	Org: BARTLETT
	MR - MR Abdomen w Contrast	Acc: 526	Date: 6/25/2014 7:00:00	MRN: 10004	Org: MRIA
	No Descriptions	Acc: 520	Date: 6/23/2014 7:30:00	MRN: 10004	Org: BARTLETT
	No Descriptions	Acc: 519	Date: 6/23/2014 7:15:00	MRN: 10004	Org: BARTLETT
	No Descriptions	Acc: 518	Date: 6/23/2014 7:00:00	MRN: 10004	Org: BARTLETT
	No Descriptions	Acc: 517	Date: 6/20/2014 7:15:00	MRN: 10004	Org: BARTLETT
	No Descriptions	Acc: 516	Date: 6/20/2014 7:00:00	MRN: 10004	Org: BARTLETT
	MR - MR Abdomen w Contrast	Acc: 473	Date: 4/24/2014 7:00:00	MRN: 10004	Org: MRIA
	US - US AAA SCREENING	Acc: 147	Date: 10/25/2013 12:21:49	MRN: 10004	Org: MIS
	TEST, ADULT	10004	1/1/1974	NBMC	M
	TEST, ADULT	10004	1/1/1974	RAPC	M

Linking Patients:

Linking Patients is performed by selecting two records (Hold Ctrl + Click) right click and select 'Link Patients'.

The list doesn't automatically refresh to show the patients are linked. Once the link is completed, click the 'Search' button to show the newly linked records.

	TEST, ADULT	10004			
	TEST, ADULT	10004			
	TEST, ADULT	10004			
	TEST, ADULT, X	16308			
	TEST, ADULT, X	16308			
	TEST, ADULT, X	16308			

Export via DICOM
 Create Patient
 Edit Patient
 Delete Patient
Link Patients
 Unlink Patient(s)

This shows two records immediately after the Patient Linking occurred without refresh/Search:

Patient Name		MRN	Date of Birth	Organization	
test, adult					
	TEST, ADULT	10004	1/1/1974	WMC	M
	TEST, ADULT	10004	1/1/1974	RAPC	M

This shows one record after hitting 'Search' refreshing the view:

Patient Name		MRN	Date of Birth	Organization	
test, adult					
	TEST, ADULT	10004	1/1/1974	WMC	M

Unlinking Patients:

An organizations record cannot be unlinked specifically.

Unlink records until the organization's record is unlinked, and then re-link the records.

Patient Name		MRN
test, adult		
	TEST, ADULT	10004
	No Descriptions	5281
	No Descriptions	5245
	No Descriptions	5244
	OT - DEXA Bone C	5008
	CR - XR Ankle Rig	5007
	RF - XR Angio Ext	5006
	CR - XR Abdomen	ACC: 4842

Export via DICOM
 Create Patient
 Edit Patient
 Delete Patient
Unlink Patient(s)
 Merge Patients
 Create Exam